

ENDOGLIDE™ TRAINING

Checklist

Before your first pre-loaded DSAEK surgery using the EndoGlide™ Ultrathin, please review the following training resources, handling procedures and online videos to ensure the best outcome for your patients:

Tissue resources

- Pre-loaded DSAEK tissue specification options
- EndoGlide™ Ultrathin handling and preparation for use
- EndoGlide™ Ultrathin product information and instructions
- Surgical pearls by Professor Donald Tan
- Preparation and insertion animation
- Live surgical demonstration video
- EndoGlide™ system loading procedure video
- Outcomes of DSAEK using eye bank-prepared preloaded grafts
- A prospective study comparing EndoGlide and Busin Glide insertion techniques in DSAEK

Handling procedures

Upon receipt of the tissue container, remove the sealed tissue container vial from the Styrofoam shipping container, or surgical facility refrigerator, 1-2 hours prior to surgery to reach ambient temperature, prior to beginning the surgical procedure.

At the time of surgery, aseptically remove the EndoGlide™ Ultrathin cartridge containing the graft from the tissue container and immerse in a BSS+ bath for 10 minutes, immediately prior to transplantation.

Contacts

Eversight

Diane Hollingsworth
Vice President, Partner Relations
(312) 469-5565
dhollingsworth@eversightvision.org

Coronet Medical

David Brown
Vice President Sales and Marketing
(610) 777-2475
dave.b@coronetmedical.com